## User Manual

### VSC24.1

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**EN** Before using, please read this manual completely and save it for future use. On our website you will find the user manual for the Streamcheck App.

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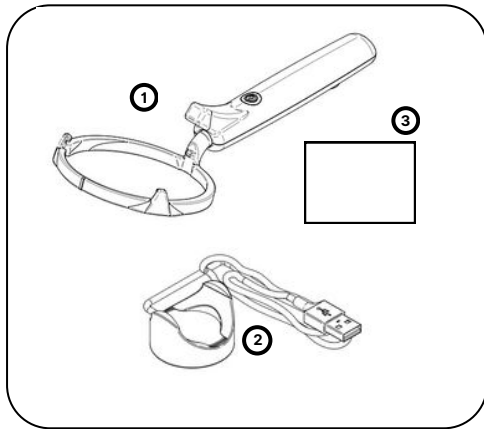
**EN**

- Streamcheck test device incl. weighing cell and camera (for measuring urine flow) x1
- Charging cable with support for inductive charging x1
- Instructions manual x1

#### Additional required equipment for use:

- Urine collection cups
- Smartphone (Android (version 11+) or iOS (version 16+))
- Streamcheck app (available in Apple Store and Google Play)
- Power supply CE Certified adapter with USB-A port 5 V | 1000 mA

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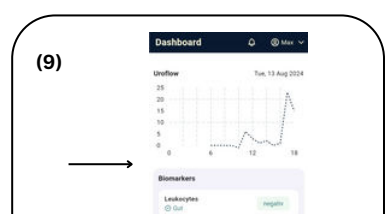
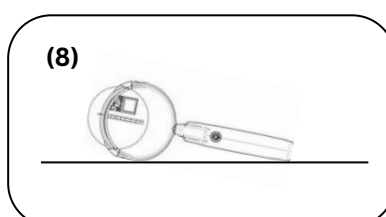
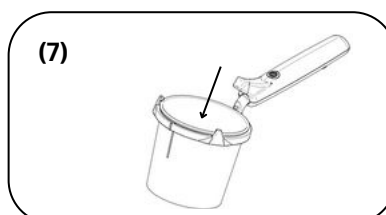
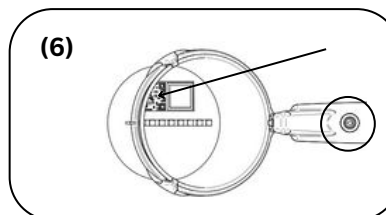
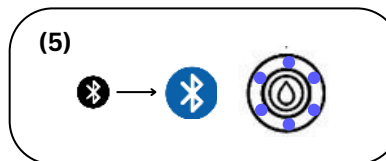
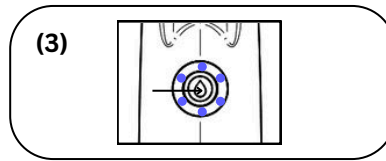
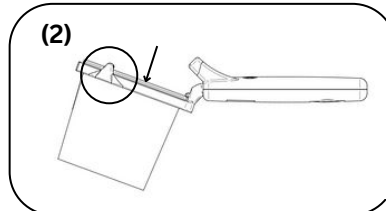
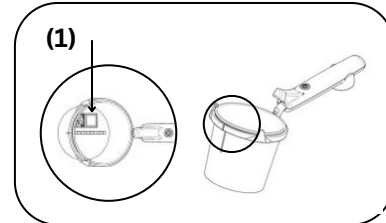
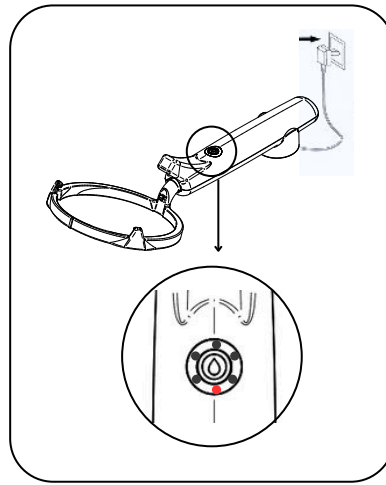
**EN** **All information in this part is provided to instruct you to use the appliance in a correct and safe way so that you can avoid injuries to yourself as well as damage to property.**

- Always read the instructions for use and the user manual in full before use.
- If you have any health concerns regarding the measurement results of the Streamcheck, consult a urologist. This device is used to record medical data. No diagnoses are made.
- Use of the device does not replace medical consultation and treatment. Therefore, always consult your doctor first if you experience any type of pain or illness.
- Observe the general hygiene measures when using Streamcheck.
- Keep children away from the packaging material (risk of suffocation).
- Do not charge it unattended and remove the charger when the Streamcheck device is fully charged to maximise battery life and minimise the risk of malfunction.
- Do not immerse the Streamcheck device in water AND protect the device from strong impact and vibration.
- Ensure that the device is operated at a sufficient distance from sources of electromagnetic radiation, such as WLAN routers, mobile phones, microwave ovens and other radio devices. A minimum distance of 1.5 metres should be maintained to ensure the proper functioning of the Streamcheck device
- If the Streamcheck device has been dropped or is visibly damaged, it shall no longer be used. If in doubt, contact customer service or your distributor.
- The Streamcheck may only be operated with the corresponding cups. The use of third-party cups will result in incorrect measurements and may damage the device.
- Only use the cup with urine test strips once. The urine test strips only work the first time they are used and cannot provide any results thereafter.
- Only use the device: on people, for the purpose for which it was developed and in the manner specified in these instructions for use.
- Under no circumstances may you open or repair the device, as otherwise its proper functioning can no longer be guaranteed. Failure to do so will invalidate the warranty.
- The appliance is maintenance-free.

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**EN** **Charging**

- Use the charger provided to charge the Streamcheck.
- The Streamcheck is charged inductively.
- To charge, place the back side of the handle of the Streamcheck on the charging cradle.
- The charging process starts when the LED display lights up orange. When fully charged, the LED display will switch off.
- The device's battery level is also displayed in the app.
- The full charging process takes about 1.5 hours with a power supply of 1000mA.
- The charging status can also be seen from the LED display.

**Note: See point 4 LED display for more information about the LED display**

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**EN**

**(1)** Insert the cup into the Streamcheck device so that the mark in the cup is aligned with the marked in the Streamcheck ring. The QR code will be then on the right side.

**(2)** The QR code is then on the right side. Now press the cup under the holding clamps. To do so, apply slight counter-pressure to the ring holding the cup.

**(3)** Start the Streamcheck by touching the Streamcheck logo on the device for about 2 seconds. Open the 'Streamcheck' app on your smartphone.

**(4)** Click "Add measurement" in the app and the app will start pairing the device. The process is displayed in the app. The completed pairing is indicated by a solid blue LED light (see 04. LED displays).

**(5)** Wait until the Streamcheck is paired (see 04. LED displays).

**(6)** The QR code on the cup is automatically read in (detected by the camera) and stored in the app for single use. The LED light will display a white running light as confirmation (see 04. LED displays).

**(7)** Urinate into the cup. Make sure that nothing is lying on the cup or that the cup itself isn't lying on something and influencing the measurement of the urine stream.

**(8)** Empty the cup by tipping it out and place it on its side on a firm surface. When placing the cup, avoid direct sunlight.

**(9)** Once the measurement is complete, the results can be viewed in the smartphone app.

**(10)** To remove the cup from the Streamcheck device, crumble the cup in your hand until the circumference is smaller than the ring and slowly slide it out of the device. Finally, dispose of the used cup as advised on the chapter "05. Cleaning and Disposal", and clean the cup ring.

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