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Instructions for use

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Modell-Nr.: 4262454270002

D-015 Revision: 03

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
D-015 Revision: 03

Publication date: 2026-03-09

Please read these instructions completely before use and keep them for future reference. You can find the user manual for the Streamcheck app on our website.

Package contents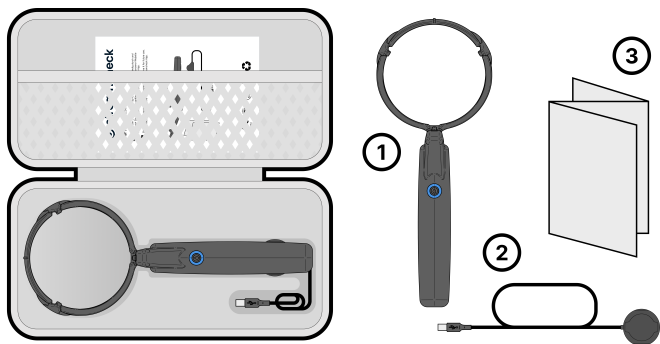

1. Streamcheck Uroflowmeter
2. Charging cable including pad for inductive charging
3. Instructions for use

Required accessories for use

- Urine specimen cups
- Smartphone Android (Version 11+) / iOS (Version 16+)
- Streamcheck-App (Download Apple App Store / Google Play)

- CE-certified power supply with USB-C port 5 V | ≥ 1000 mA

1. Intended purpose and intended users

Streamcheck is a medical self-testing device for men aged 18 and over for conducting urine tests. It serves for the long-term monitoring and assessment of urinary tract health through repeated measurements. The app generates structured PDF reports that can be shared with doctors. The device itself does not make diagnoses or interpret data, but provides reliable measurements to support clinical decisions.

Main functions

- Urine flow monitoring: Recording of urine flow patterns over extended periods with a maximum deviation of $\pm 10\%$ to assess bladder function.
- Biomarker analysis: Evaluation of standardized urine test strips located inside at the bottom of a disposable urine specimen cup to detect abnormalities in the urinary tract and beyond.
- Anomaly detection & guidance: The app automatically analyzes measurement data. In case of anomalies (e.g., low Qmax or abnormal biomarkers – color-coded green, orange, red, gray), it provides specific instructions, such as to repeat the test or to visit a doctor.
- Data output: After each measurement, the app generates a structured PDF report with uroflow curve, biomarker values, timestamps, and Traceability data. The report is intended solely for evaluation by medical professionals and does not contain diagnoses or automated assessments.

Contraindications

The Streamcheck measuring device must not be used by the following persons:

- with severe cognitive impairments that make it impossible to understand and follow the instructions for use – unless the device is used under supervision.
- with severe motor impairments (e.g. paresis, severe tremor) that make safe and effective handling of the device impossible.

- with known allergies to materials used in the device housing that come into direct contact with the skin (e.g. ABS or MABS plastics).

2. Safety instructions and warnings

All information in this section is intended to instruct you to use the device correctly and safely to use so that injuries and property damage can be avoided. If you have any questions, contact the manufacturer.

- Always read the instructions for use and the user manual completely before use.
- Adherence to the operating instructions is mandatory to ensure proper and effective operation.
- If you have health concerns regarding the Streamcheck measurement results, consult a urologist. This device is for collecting medical data; it does not make diagnoses.
- Using this device is not a substitute for medical advice or treatment. Therefore, always consult your doctor first if you experience any kind of pain or illness.
- Please observe general hygiene measures when using Streamcheck.
- Do not leave the device charging unattended and remove the charger once the Streamcheck measuring device is fully charged to maximize battery life and minimize the risk of malfunctions.
- Do not immerse the Streamcheck measuring device in water and protect it from strong shocks and vibrations.
- Keep children away from the packaging material (risk of suffocation).
- Ensure the device is operated at a sufficient distance from sources of electromagnetic radiation – such as Wi-Fi routers, mobile phones, microwave ovens, or other radio devices. A minimum distance of 1.5 meters should be maintained to guarantee proper functioning of the Streamcheck measuring device.
- If the Streamcheck measuring device has been dropped or shows visible damage, it must not be used. If in doubt, please contact customer service or your retailer.
- Any serious incident relating to the device must be reported to Stream-

check GmbH and the competent authority of the EU Member State in which the device was purchased.





- The Streamcheck measuring device must only be used with the urine specimen cups provided for this purpose. Using other urine specimen cups will lead to inaccurate measurements and may damage the device.
- Use the urine specimen cup with urine test strips only once. The urine test strips only work the first time they are used and will not provide usable results afterward.
- Use the device only on humans, for its intended purpose and in the manner described in these instructions for use.
- Under no circumstances should you open or attempt to repair the device yourself, as this will compromise its proper functioning. Failure to comply will void the warranty.
- This device must not be modified without the manufacturer's permission. If modified, the manufacturer can no longer guarantee the device's safety.
- The device is maintenance-free.
- There is a risk of measurement inaccuracy due to incorrect positioning (e.g., if the penis rests on the rim of the urine specimen cup).
- There is a risk of malfunction due to external influences or environmental conditions such as magnetic fields, external electrical and electromagnetic interference, electrostatic discharges, radiation associated with diagnostic or therapeutic procedures, pressure, humidity, or temperature. Therefore, always store the device in the supplied case.
- Measurements with a total urine volume below 150ml may not provide valid uroflowmetry results; the app will inform the user accordingly in this case.
- If symptoms indicate an acute urinary tract infection (e.g., fever, abdominal pain), it is recommended to consult a doctor before using the device.

3. Operating Instructions


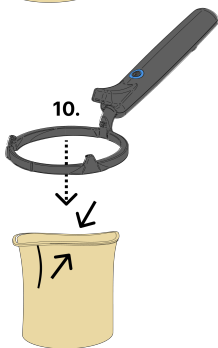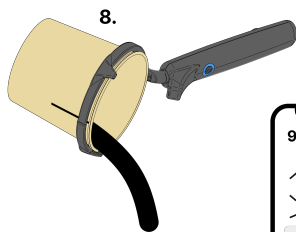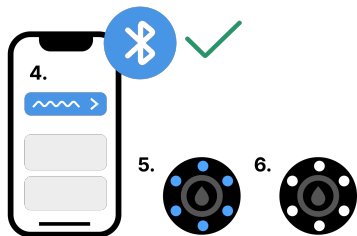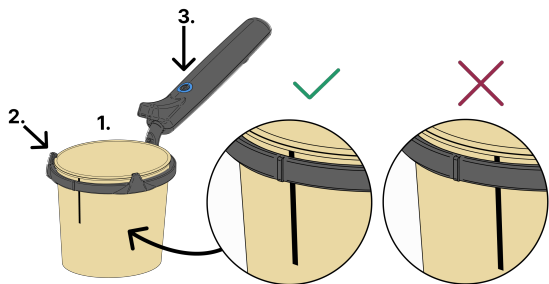
1. Download the Streamcheck App (Apple App Store / Google Play)
2. Place the urine specimen cup in the Streamcheck measuring device so that the mark on the urine specimen cup aligns with the mark on the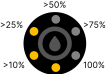

Streamcheck ring. The QR code and biomarker test strip are located inside at the bottom of the urine specimen cup.

3. Now press the urine specimen cup under the retaining lugs of the urine specimen cup ring. Apply gentle counter-pressure to the underside of the urine specimen cup ring.
4. Activate the Streamcheck measuring device by touching the Streamcheck logo on your device for approximately 2 seconds. Open the Streamcheck app on your smartphone.
5. Add another measurement in the app to start the device pairing process. The process will be displayed in the App. Successful pairing is indicated by a solid blue LED light (see diagram 4. LED display, Fig. 10).
6. The QR code located inside at the bottom of the urine specimen cup is automatically scanned by the app using the camera and stored for one-time use. The app then confirms whether the urine specimen cup has been correctly identified and initiates the measurement. The app screen displays when you can begin urinating into the urine specimen cup.
7. Only urinate into the urine specimen cup when the app prompts you to. Ensure that nothing is resting on top of the urine specimen cup during the measurement and that the urine specimen cup itself is not placed on any surface, as this can affect the measurement of the urine stream.
8. Next, empty the urine specimen cup by tipping it out and place the Streamcheck measuring device on its side on a firm surface. Data transmission is indicated by a white LED light (see diagram 4. LED display, Fig. 6). Avoid direct sunlight when positioning.
9. The measurement results can then be found in the smartphone app.
10. To remove the urine specimen cup from the Streamcheck measuring device, gently squeeze the urine specimen cup until its circumference is smaller than the ring. Slowly slide it out of the device. Finally, dispose of the used urine specimen cup (see section 5. Cleaning and Disposal).
11. After successful analysis of the measurement and data transmission, the device is automatically switched off.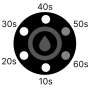

4. LED display

Nr	Led	Description
1		Lights up for 2 seconds Battery level indicator at start
2		Rapid flashing Low charge level (no measurement possible)
3		The red light under the LED ring shines constantly. Charging process
4		The red light under the LED switches off. Charging process complete
5		Lighthouse Effect Measurement / Uroflow process
6		Staggered LED illumination Refilling the LED ring after the uroflow process / reaction time
7		Rapid flashing Critical error (Sleep mode will be activated after 10 seconds)
8		Slow blinking Warning (User action is required)
9		Blinking Ready for Bluetooth pairing with the mobile device

Nr	Led	Description
10		Lit up Bluetooth pairing successful
11		Blinking Update is being performed

5. Charging process

- Use the provided charger to charge the Streamcheck measuring device combined with a CE-certified power adapter with a USB-C connector, 5V | 1000mA.
- The charger can remain permanently in the storage case. This allows the Streamcheck measuring device to charge in the case without having to remove the charger. At the same time, ensure that the device is always correctly positioned.
- The charging process begins when the LED indicator lights up red. Once the device is fully charged, the LED indicator switches off.
- The device's charge level is also displayed in the app.
- The full charging process takes approximately 1.5 hours.
- The charge level is also visible via the LED display (see 04. LED display).

6. Cleaning and disposal

Cleaning and disinfection

The outer casing of the Streamcheck measuring device and the glass cover of the camera must be carefully cleaned and disinfected after each use. Do not immerse the device in liquid. Disinfection with UV radiation is not permitted.

Disposal

The product must be disposed of in accordance with the local waste disposal laws. Electronic components must be disposed of in accordance with

the regulations for waste electrical and electronic equipment.

Packaging disposal

The plastic outer packaging of the urine specimen cups should be disposed of in the yellow bin.

Urine specimen cup disposal

The used urine specimen cups must be disposed of in the residual waste.

Disposal of old appliances

In the interest of environmental protection, this device must not be disposed of with regular household waste at the end of its life. Disposal can be carried out via appropriate collection points in your country. Dispose of the device in accordance with the EU Directive on Waste Electrical and Electronic Equipment (WEEE). If you have any questions, please contact the relevant waste disposal authority.

7. Troubleshooting

The following are possible errors and their solutions related to the use of the Streamcheck measuring device.

Problem

The device reverts to pairing mode after use. The device has restarted.

The device does not turn on after touching the Streamcheck logo.

Solution

Wait approximately 2 minutes, the device will automatically go into sleep mode.

Charge the device.

Problem	Solution
The device will not pair with the smartphone.	Close the app, re-enable Bluetooth, and restart the app.
The urine specimen cup cannot be detected.	Check that the urine specimen cup is correctly aligned so that the mark on the urine specimen cup matches the mark on the ring of the Streamcheck measuring device.
The app cannot connect to the device.	Turn Bluetooth off and then on again on your mobile phone. Make sure the Streamcheck measuring device is charged. See LED indicator 4, line 2.

Please contact customer support if there are still some questions or uncertainties.

8. Declaration of Conformity

This product complies with the requirements of the EU Medical Device Regulation (MDR) 2017/745. The undersigned, acting on behalf of the manufacturer, hereby declares that the product described above complies with the relevant requirements of the EU Medical Device Regulation (MDR) 2017/745. This product was designed, manufactured, and tested in accordance with the following EU directives and standards (EU Medical Device Regulation MDR 2017/745). The conformity assessment procedure was carried out in cooperation with the notified body:

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Number: 0494

Conformity assessment procedure: CB certificate EMC

9. Technical Specifications

- **Device type:** BF
- **Dimensions (B,L,H) :** 125 mm x 311 mm x 58 mm
- **Weight:** 105 g
- **Battery capacity:** 720 mAh
- **Battery voltage:** 3.7V
- **Charging current:** max. 750 mA
- **Maximum transmission power:** BLE (2402–2480 MHz): 20 dBm
- **Measuring range:** 0–1000 ml; 0–100 ml/s; 0–1200s
- **Guaranteed lifespan:** 60 months

All measurements in the app are given in SI units. Volume is displayed in ml (with a tolerance of +/- 5%) and urine flow in ml/s (with a tolerance of +/- 10%).

Protection class

- **Device:** IP32
3x: Protection against solid objects with a diameter greater than 2.5 mm;
x2: Protection against vertically falling water when the device is tilted by 15 degrees.
- **Charging adapter:** IP21
2x: Protection against solid foreign objects with a diameter of more than 12 mm; x1: Protection against vertically dripping water

Open Source Licenses Used

- Standard C Libraries — ISO/IEC 9899:2018
- ESP-IDF (Espressif IoT Development Framework) — 1.8.0
- FreeRTOS — V10.4.3
- LSM6DSOX Driver — V3.0.1

Environmental conditions

- **Temperature:** 0 °C – 30 °C
- **Humidity:** 10 % – 90 %
- **Air pressure:** 700 hPa – 1060 hPa

10. Manufacturer's specifications

Manufacturer:

Streamcheck GmbH

Address:

Streamcheck GmbH

Arnsdorf 26

02894 Vierkirchen

Germany

E-mail: support@streamcheck.io

Website: www.streamcheck.io

IFU

Website

App

11. Symbols

Symbol

Description

Manufacturer

Date of manufacture

Serial number

Protect from moisture

Symbol	Description
	Fragile
	Read the instructions for use.
	CE certification
	Limiting atmospheric pressure
	Limiting humidity
	Ambient temperature

Disclaimer: This content has been translated from German into English using automated tools. While we strive for accuracy, please be aware that the original German version is the official and binding document. Any discrepancies in the translation are not legally binding.

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